
# HEINE mini-c® Cliplampe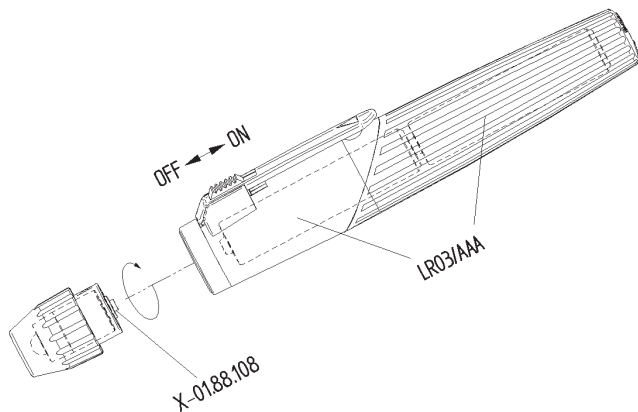

## HEINE

HEINE Optotechnik GmbH & Co. KG  
Kientalstr. 7 · 82211 Herrsching · Germany  
Tel. +49(0)81 52/38-0 · Fax +49(0)81 52/38-202  
E-Mail: info@heine.com · www.heine.com  
med 3906 1/5.07

CE 93/42/EWG/CEE/EEC

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DEUTSCH

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## Instructions

ENGLISH

 In accordance with local regulations this product should be disposed of as an electronic device separately.

Read these instructions carefully before using the HEINE mini-c® Cliplamp and keep them in a safe place for future reference.

**Range of applications:**  
The HEINE mini-c® Cliplamp is designed solely for the examination of the surface of the skin.

**Please note: The performance of this instrument can only be guaranteed if genuine HEINE Accessories are used.**

**Cleaning:**  
The Cliplamp can be cleaned with a damp cloth (e.g. alkaline or pH-neutral detergent).  
For cleaning disinfection of the outer surface we recommend the so-called **wipe disinfection** by using detergents, which are released for Medical Devices made out of plastic.  
Disinfection by spraying or immersion as well as sterilisation is not allowed.

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## Warranty

We guarantee this device (except bulbs) for five years from date of purchase. We guarantee the proper function of the unit provided it is used as intended by the manufacturer and according to the instructions for use. Any faults or defects which occur during the guarantee period will be repaired free of charge provided they are caused by faults in material, design or workmanship. In the case of a complaint of a defect in the product, the buyer must prove that any such defect was present at the time the product was delivered. This guarantee excludes any defects which occur because of wear and tear, incorrect use, the use of non-original HEINE parts, (especially bulbs, as these are designed with emphasis on the following criteria: Color temperature, life expectancy, safety, optical quality and performance). Also excluded are repairs or modifications made by persons not authorised by HEINE or cases where the customer does not follow the instructions supplied with the product. Further claims, in particular claims for damage not directly related to the HEINE product, are excluded.

For repairs, please contact your supplier, who will return the goods to us or our authorized agent.

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