# Fotofix

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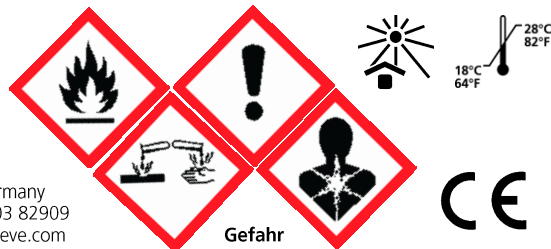
## Directions for use

### Product description:

Fotofix is a glue especially for ITE housing shells made of Fotoplast® for bonding with a faceplate. It bonds two parts reliably; it can, however, be detached with a scalpel for repairs of the ITE hearing aid.

### Processing:

Apply Fotofix on faceplate or Fotoplast®-shell, put together both parts and - perhaps - allow Fotofix to flow out of the shell from the outside.

**Curing time:** appr. 10 min. in the Polylux

**Storage:** 18 – 28°C, secure top after use, protect from light unconditionally!

### Hazard statements

- H225 Highly flammable liquid and vapour.
- H315 Causes skin irritation.
- H318 Causes serious eye damage.
- H317 May cause an allergic skin reaction.
- H361f Suspected of damaging fertility.
- H335 May cause respiratory irritation.
- H412 Harmful to aquatic life with long lasting effects.

### Precautionary statements

- P210 Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking.
- P303+P361+P353 IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water/shower.
- P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.
- P310 Immediately call a POISON CENTER/doctor.
- P405 Store locked up.
- P501 Dispose of contents/container in accordance with local/regional/national/international regulations.

**Contains:** methyl methacrylate, 2-Propenoic acid, reaction product with Pentaerythrite, Diphenyl(2,4,6-trimethylbenzoyl)phosphinoxide, Acrylic resin

**Storing time:** 2 years

Dreve Otoplastik GmbH is not liable for any damages caused by improper application of the material.

2016-01-21-ak/ss/es/vb  
Dreve Otoplastik GmbH  
Max-Planck-Str. 31 • 59423 Unna / Germany  
Tel.: + 49 2303 8807-0 • Fax: + 49 2303 82909  
E-Mail: otoplastik@dreve.de • Internet: www.dreve.com